

High Top Air Cam Walker Boot

INSTRUCTIONS SHEET:

INDICATIONS FOR USE:

- Acute Ankle Sprain
- Stress Fractures of the Lower Leg/Ankle
- Stable Delayed Union or Non-Union Fractures of the Distal Tibia and Fibula
- Achilles Tendon Repairs or Ruptures
- Soft Tissue Injuries
- Syndesmosis (High-Ankle Sprain)
- Post-ORIF and Surgical Stabilization
- Stable Foot/Ankle Fractures
- Forefoot/Midfoot/Hindfoot Pain and Injury
- Post Bunion Surgery
- Post Tendon/Ligament Repairs
- Tarsal, Metatarsal, and Phalange Fractures

CLEANING INSTRUCTIONS:

- Hand wash with mild detergent.
- Rinse and air dry.
- Do NOT use bleach.
- Do not expose to direct heat or sunlight while drying to prevent shrinkage and damage to your support.



INSTRUCTIONS FOR USE:

Before applying the brace, please read these instructions completely and carefully. Correct application is vital to the proper function of the braces.

- For first time application, loosen straps and remove the liner from the boot.
- Place foot in the liner and secure with contact closure. Fasten the foot flaps on the liner first, making sure they are snug. Wrap and fasten the leg portion of the liner, making sure that it is snug from the bottom to the top.
- Some trimming of the liner may be necessary to accomplish optimum fit.
- Spread the uprights using both hands and step into the boot.
- Remove the cardboard piece marked "PULL UP" to affix the upright to the liner.
- Place the straps through the matched slits, and secure boot straps starting at the toes and working up the leg.

Foam pads are provided to alleviate the problem of excess rubbing. These pads can be placed inside the liner or between the liner and the uprights.



PLEASE NOTE:

The High Top Air Cam Walker Boot is designed for support and does not guarantee injury prevention. By using this product, consumers or patients acknowledge and agree to the following:

Innovative DME is not responsible for any injuries that may occur during the use of Innovative products.

Skin irritation is a potential side effect when using this device. If you experience an unusual skin reaction, it is essential to consult with your healthcare provider immediately.

Always consult your healthcare provider to ensure that this product is suitable for your specific medical condition and needs.

This product is intended for use by a single patient only.

Federal Law restricts the sale or distribution of this device to be carried out by or under the order of a licensed healthcare practitioner.

This product was designed to complement the variety of medical treatments common to the above afflictions. The frequency and duration of use should be determined by your prescribing healthcare professional.

If you do not agree with these conditions, please contact your healthcare provider before using this product.



sales@innovatedme.com



(352) 587-7613



16034 US 19 Hudson, FL 34667