

Urine Analyzer



**New product is
more precision
and reliable !**

MEDIX Urine Analyzer

[Measurement Principle] : Artificial Intelligence + computer vision technology

[Scope of Application] : 10,11,12 and 14 parameters

[Test Items] : Glucose (GLU), Billirubin (BIL), Ketone (KET), Specific gravity (SG), pH, Blood (BLD), Protein (PRO), Urobilinogen (URO), Nitrite (NIT), Leukocyte (LEU), Ascorbic acid (VC), Creatinine (CRE), Calcium (CAL) and Microalbumin (MAL)

[Test Speed] : 514 samples/hour

[Record Method] : Thermal print/LCD display

[Display] : 7-inch large screen display, touch operation

[Voltage] : AC 100~240V \pm 10%, 50-60Hz, safe and reliable

[Working Environment] : 15-35 $^{\circ}$ C, Humidity <85 %

[Power] : 20VA

[Net Weight] : <5KG

[Data Communication] : Reserved RS232 interface, WIFI, network interface, external urine sediment analysis system, hospital management software (HIS, LIS)

[Data Storage] : Large-capacity memory. Can store millions of test results, can query the test result according to the record numbers

[Printing Method] : Built-in high-speed thermal printer



Urine Analyzer

Never satisfied with present,
New product—BW-300.
More precision and reliable cause of the
computer visual technology + AI
technology.

Bioway invites you to experience the new
large touch screen feeling.

The choice of a new generation.

Urine Strips

Model (portion of models)	Applicable Machine
URS- 1	Visual
URS- 2	Visual
URS- 3	Visual
URS- 4	Visual
URS- 5	Visual
URS- 6	Visual
URS- 7	Visual
URS- 8	Visual
URS- 10H	BW-200 / BW-300 / BW-500 / Visual
URS- 11H	BW-200 / BW-300 / BW-500 / Visual
URS- 12H	BW-200 / BW-300 / BW-500 / Visual
URS- 14H	BW-200 / BW-300 / BW-500 / Visual
URS- 10F	BW-901
URS- 11F	BW-901
URS- 12F	BW-901
URS- 14F	BW-901
Package Quantity	100 Pcs //bot.



URS-10 (Visual)

Instructions:
 Holding the end of the test strip, completely immerse the test strip in the urine sample. Remove the test strip and observe the results within 1-2 minutes. Lightly wipe the test strip on the absorbent paper to remove excess urine. Waiting for 1 minute at least, reading the results. Comparing with the color chart and the color result.

Replace cap promptly and tightly

Hainan Biological Technology Co., Ltd.
 1st floor, No. 1000000
 Add: Tropical and subtropical area
 No. 1, Hainan, China

MADE IN CHINA

For Professional Use Only

Reagent strips for urinalysis

For Visual (Visual) Usage Only