



Susan Wingrove Queen of Hearts Protocols for Hygienists and Doctors

Queen of Hearts by Paradise Dental Technologies, Inc. (PDT). **Universal curette with 6mm cutting edge on both sides and toe.** Curved to adapt to all natural teeth root surfaces, furcations, line angles, and on lingual can go interproximal. Ideal for hygienists and doctors as a surgical curette.

Instrumentation: PDT Queen of Heart Universal Curette

- Insert like a probe, parallel to the root surface. Debride in a basketweave stroke, horizontal side to side motion and push/pull vertical stroke on the root surface.
- Start in posterior, insert parallel to the root surface and move side to side with horizontal stroke and vertical push / pull motion. If a furcation present, drop into furcation turn and twist to debride. Using a horizontal motion, slip around the distal molar to debride the line angle.
- Flip blade to adapt to lingual root surfaces and Queen of Hearts can go interproximal to debride furcations, and fossa of the first premolar.

Instrumentation: Surgical Applications – Doctors Use as a Surgical Curette

- To remove granulation tissue after surgical tooth extractions and prior to socket preservation.
- Debriding stringy, fibrous tissue in furcation. Guided Tissue Regeneration (GTR)
- Removes disease tissue in mesial of first premolar fossa, prior to Guided Bone Regenerative (GBR) procedure.
- Use prior to all defect GBR procedures, sinus and distal mandibular molar augmentation.