



StomaToGo Transport Instructions



StomaToGo Transport Instructions

1. Recommend transporting with a waste bag already prepared
2. Insert Bag Holder into the Base with or without waste bag
3. Raise the Stabilizer Bar
4. Wrap the belt/strap around the holders on the Base and the Stabilizer bar under the posts on the sides of the bar
5. Use the strap holding strip to hold the strap in place
6. For Basic Base skip next two step
7. Place the Shield Holder on the shield holder posts on the Base
8. Position the Rest Platform over the Bag Holder
9. Insert whole StomaToGo into a back pack or other carrier