

NOVUS+LASE NOVUS-CRYO TREATMENT PARAMETERS

ALWAYS DETERMINE THICKNESS OF FAT

Thickness of fat (CM)	Temperature (°C)	Pressure (KPA)	Time (mins)
1	0 to -1	1	10-20
2	-1 to -2	1-2	15-25
3	-2 to -3	1-2	20-30
4	-2 to -4	1-3	25-35
5	-3 to -5	2-3	30-40
6	-4 to -6	2-4	35-45
7	-4 to -7	3-5	40-50
8	-5 to -8	3-5	40-50
9	-5 to -9	4-5	40-55
10	-6 to -10	4-5	40-60

HELPFUL TIPS:


1. ON machine start-up ,you can adjust the Pressure to 10KPA, once you feel the handle being suctioned by the skin slowly lower down pressure to desire settings.

NOVUS+LASE NOVUS-CRYO TREATMENT PARAMETERS

2. Before treatment, Please apply anti-freezing membrane on the treatment area, then wrap the membrane, Lastly place handle on the desired treatment area.

Why use the Plastic Wrap?

1. To avoid clogging of the applicator hand-piece and prolong the life of your applicator. Once impurities or gel is absorbed decreased performance will be observed.

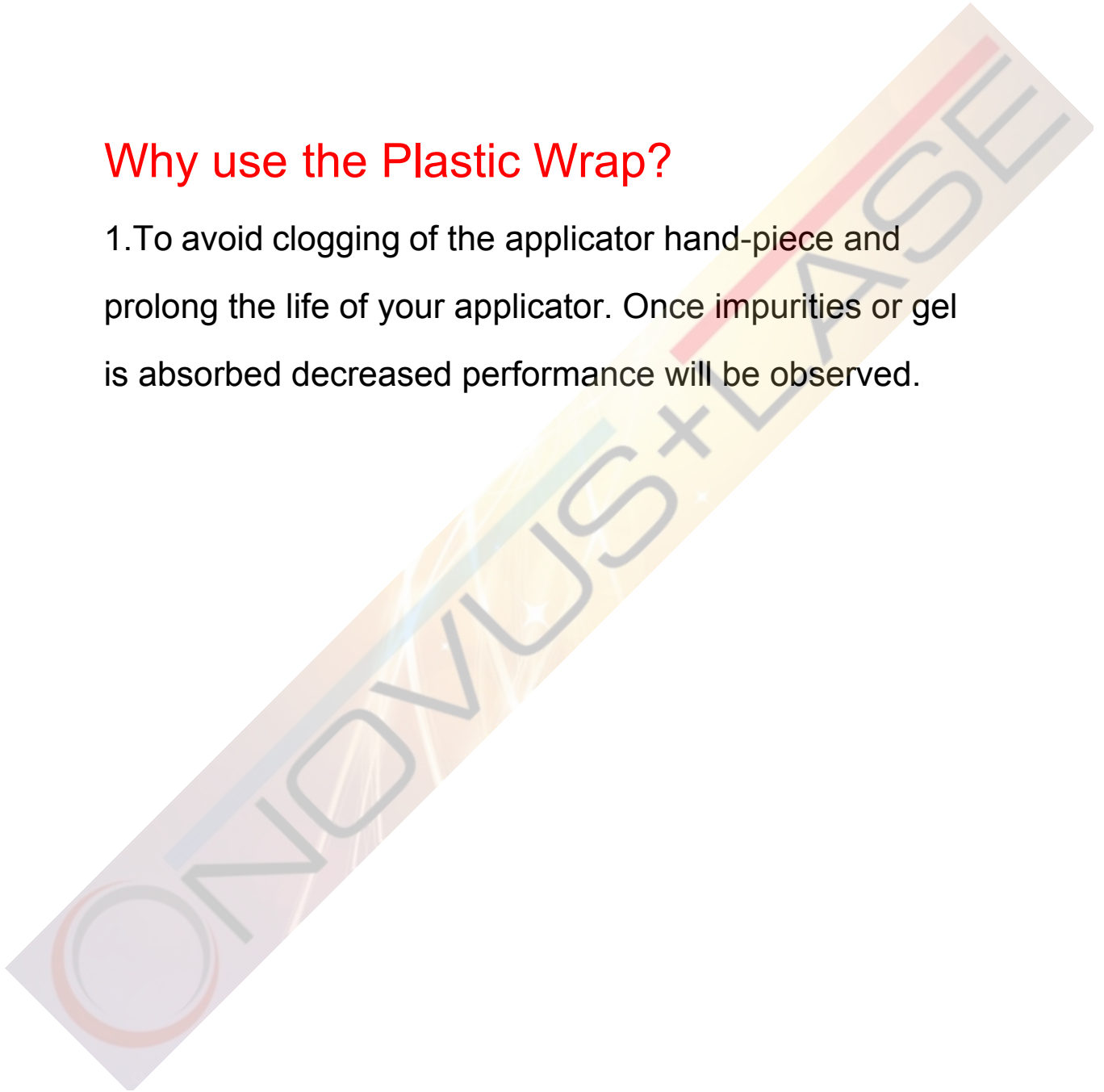

NOVUS+LASE NOVUS-CRYO TREATMENT PARAMETERS



Step 1: Place cooling membrane sheet on the desired treatment area.

Step 2: Wrap the desired area with plastic wrap around two passes.



NOVUS+LASE NOVUS-CRYO TREATMENT PARAMETERS



Step 3. Place the applicator handle on the desired treatment area.

