

# “UROMAXX™

The Pelvic Floor Chair.



# What Is UROMAXX Treatment?

UROMAXX is a cutting-edge, non-invasive electromagnetic therapy designed to strengthen pelvic floor muscles, addressing issues related to pelvic floor dysfunction. This revolutionary treatment offers a comfortable, effective solution for regaining control and improving quality of life without the need for surgery or invasive procedures.

---

## How UROMAXX Works.

UROMAXX utilizes targeted electromagnetic energy to induce powerful, supramaximal muscle contractions, stimulating the pelvic floor muscles far beyond what is achievable through voluntary effort. In a single 30-minute session, the device produces approximately 11,200 comprehensive contractions, which is clinically equivalent to performing 11,200 perfect Kegel exercises, providing rapid and significant strengthening.

# How UROMAXX Strengthens Pelvic Floor Muscles



## **Pelvic Floor Support**

Weakened pelvic floor muscles fail to adequately support pelvic organs and bladder function, leading to incontinence and other issues.

---

## **Targeted Electromagnetic Stimulation**

Thousands of supra-maximal contractions are delivered directly to the pelvic floor muscles per session, far exceeding voluntary effort.

---

## **Restored Neuromuscular Control**


Improved muscle strength and enhanced neuromuscular control leads to better pelvic floor function and regained bladder control.

# Why Patients Choose UROMAXX.



## CLINICALLY PROVEN

Validated results demonstrating efficacy and safety in patient trials.

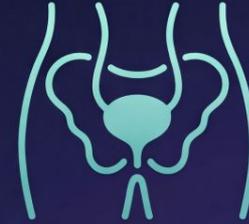

## TARGETED RELIEF

Precision solutions designed for specific urological needs.



## ENHANCED FUNCTION

Promotes better control and long-term pelvic health.



## CONVENIENT SOLUTIONS


Easy-to-integrate regimens for daily lifestyle.

# Pelvic Floor Dysfunction: Patient Symptom Spectrum.




## Urinary & Bowel Symptoms

- Stress incontinence
- Urge incontinence
- Nocturia
- Postpartum leakage
- Bowel urgency

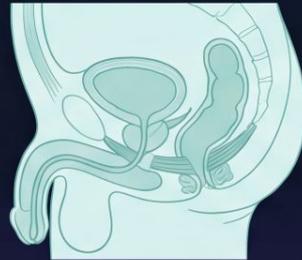


## Sexual Health Concerns

- Erectile dysfunction
- Premature ejaculation
- Decreased sensation
- Vaginal laxity
- Reduced orgasmic intensity

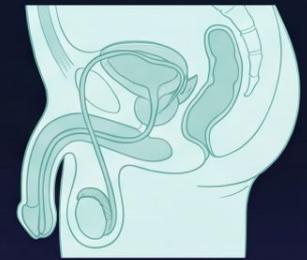


## Low Back & SI Joint Pain

- Chronic L4-S1 discomfort
- Sacroiliac instability
- Anterior pelvic tilt
- Postural imbalance

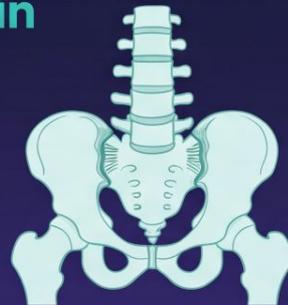


## Core Instability

- Weak transverse abdominis
- Postpartum core weakness
- Functional instability
- Reduced athletic performance

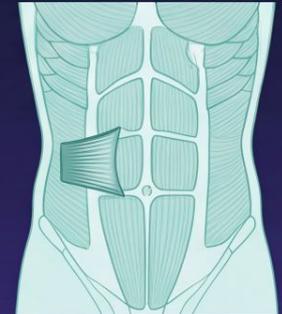

# Why Pelvic Floor Weakness Impacts the Entire Core System.

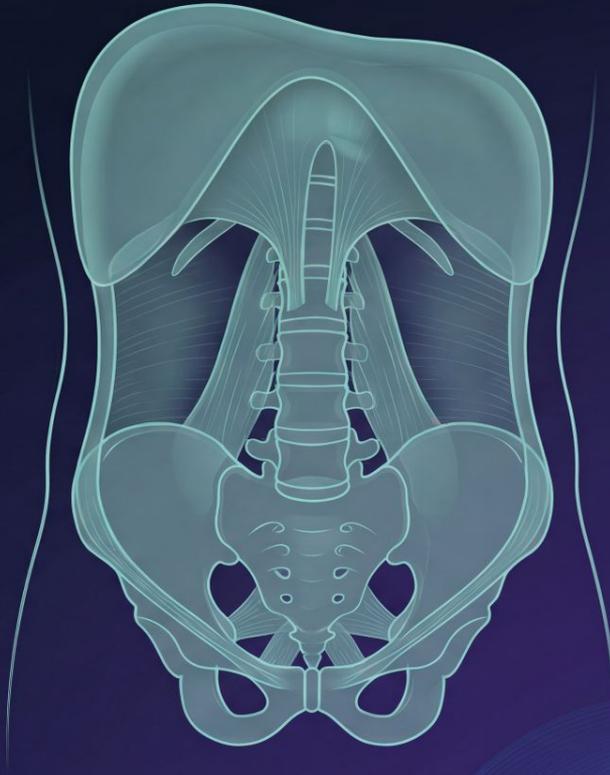

---

## Loss of Lumbopelvic Stability

- Pelvic floor forms the base of the core system
- Weakness increases stress on lumbar discs and SI joints

---

## Reduced Neuromuscular Coordination

- Disruption between diaphragm, transverse abdominis, and deep spinal stabilizers
- Impaired core activation and load transfer


---

## Bladder & Sexual Dysfunction

- Reduced sphincter support
- Altered blood flow and muscular control

# UROMAXX Operational Framework

How the Technology Engages the Pelvic Floor

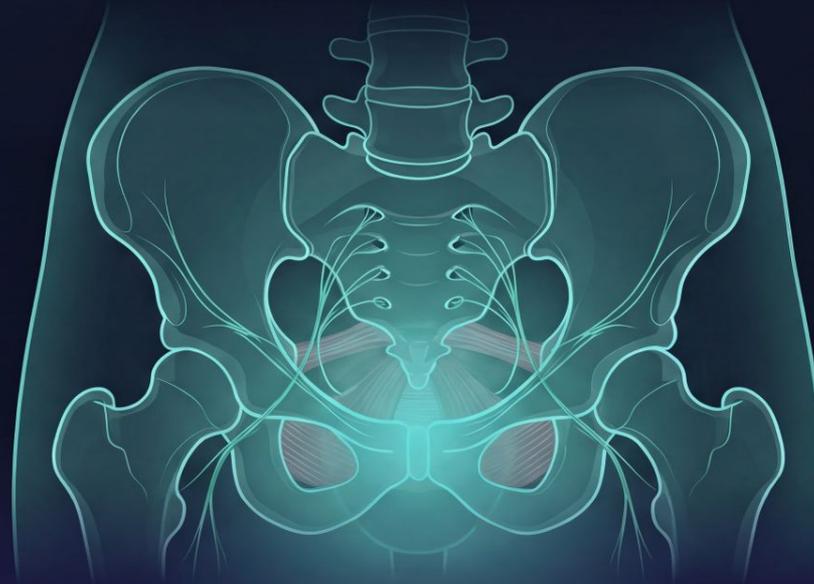

## 01 Energy Penetration

Pulsed electromagnetic waves pass through clothing and tissue, reaching deep pelvic floor structures without surface discomfort.

## 02 Neuromuscular Activation

Motor nerves are stimulated, triggering strong involuntary contractions beyond voluntary effort.

## 03 Progressive Strengthening

Repeated contractions re-educate muscles, build endurance, and support long-term pelvic stability and control.

**Treatment Duration:** ≈ 30 Minutes

**Session Frequency:** 5–8 Sessions (Twice Weekly)

**Therapeutic Focus:** Pelvic Floor Activation & Control



# Regain Control. Restore Confidence.

UROMAXX Pelvic Floor Therapy



# Why Traditional Pelvic Floor Methods Often Fall Short.

## Home-Based Kegel Exercises

- Up to 1 in 4 women perform Kegels incorrectly
- Inconsistent adherence limits measurable results
- Voluntary contractions lack depth and intensity



## Pelvic Floor Physical Therapy

- Referral completion rates remain low
- Many patients discontinue before meaningful progress
- Time-intensive and compliance-dependent



# What Patients Can Expect During Treatment.



## During the Session



Patients may notice a mild tingling sensation and rhythmic pelvic muscle contractions. The treatment is comfortable and non-invasive.

## Treatment Protocol

6-8 sessions  
30 minutes per session  
Twice weekly for 4 weeks

## After Treatment

Patients resume normal daily activities immediately following each session.

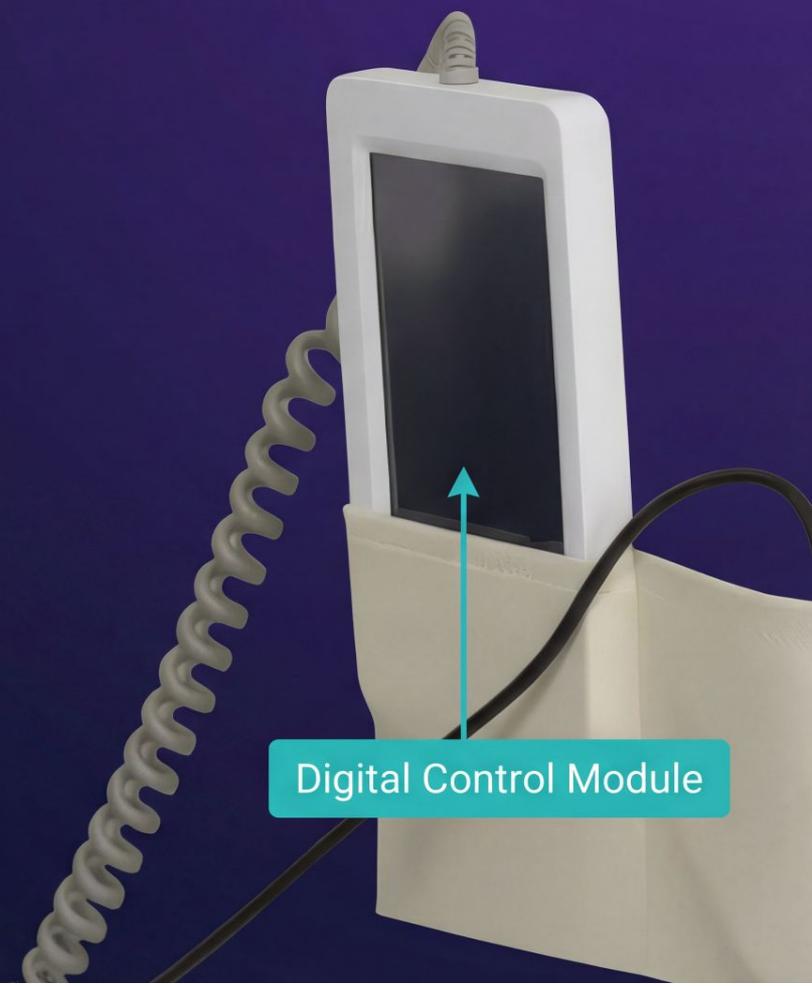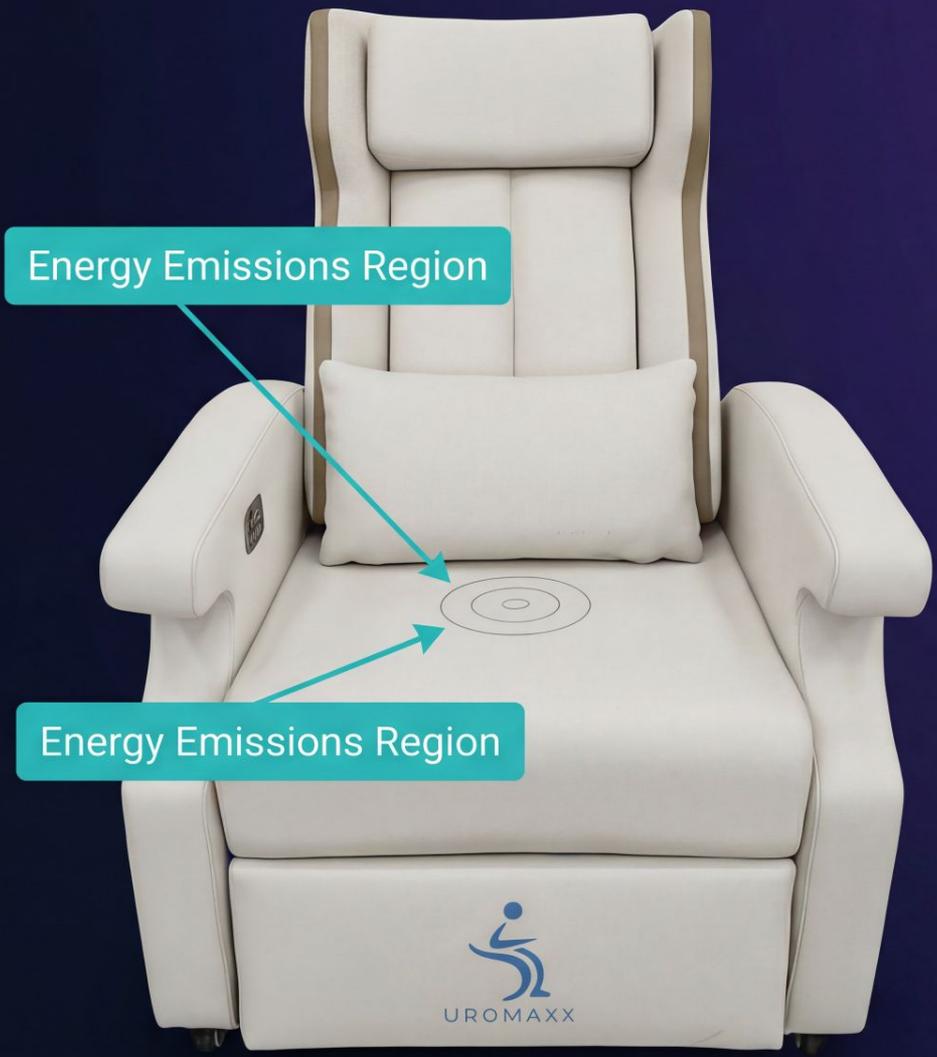



# Treatment Guide



Program Selection



Gender Selection



Treatment Mode



Intensity Control

# Patient Eligibility & Safety Considerations.

## Pregnancy

Treatment is not recommended during pregnancy.

## Metal Implants

Patients with certain metal implants in the pelvic region should undergo evaluation prior to treatment.

## Cardiac or Tumor Conditions

Individuals with specific heart disorders or active tumors require medical screening before therapy.



# Why Choose UROMAXX?

Advanced Non-Invasive Technology.

## Non-Invasive Technology

Utilizes electromagnetic energy to activate deep pelvic floor muscles without physical exertion or disrobing.

## Targeted Stimulation

Designed to enhance neuromuscular control, improve bladder function, and support core stability through focused pelvic floor activation.

## Comfort & Convenience

Fully clothed sessions in a relaxed seated position, providing a discreet and comfortable experience.

## Flexible Scheduling

1-2 sessions per week, 30 minutes per session,  
Minimum 48 hours between treatments.

## Multiple Protocols

Customizable treatment settings for varying levels of pelvic floor dysfunction in both men and women.

## Ease of Use & Reliability

Intuitive operation with minimal oversight required.  
Built for durability in professional clinical environments.



# UROMAXX

THE FUTURE OF PELVIC FLOOR THERAPY



**Strength. Stability. Confidence.**

**(212) 347-8760**