



Susie & Max Haidsiak
Reproduction Specialist

(712) 621-9830
healvetusasusie@yahoo.com

Healvet®

Veterinary Immunofluorescence Quantitative Analyzer Healvet 3000 Machine Progesterone Test Operation Procedure



1



Turn on the Healvet Analyzer

2




Insert the ID chip,
read ID chip info.

3



INPUT PET INFORMATION!

4

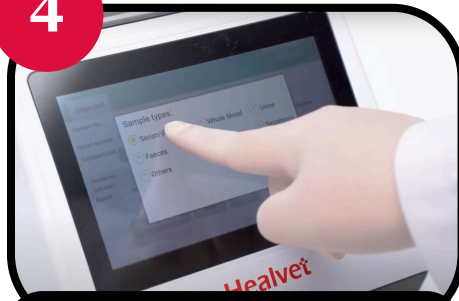

Choose sample type

5



Adjust the range of pipette to 75ul

6



Use pipette to draw 75ul of the
sample

7



Add the sample to the detection
buffer tube

8



Mix the sample with buffer by
pipetting for 4-5 times

9



Gently mix the sample well



Susie & Max Haidsiak
Reproduction Specialist

(712) 621-9830
healvetusasusie@yahoo.com

Healvet®

Veterinary Immunofluorescence Quantitative Analyzer Healvet 3000 Machine Progesterone Test Operation Procedure

10



Draw 75ul of the sample mixture

11



Add the mixed sample into the test strip

12



Insert the test strip into the analyzer

13



Time counting down, after ~15 minutes read and test result displayed on the analyzer screen.

14



Press Print to print test result

15



Your test is complete!



Still have questions or want to follow along on video?

Scan the QR Code to follow along!