

Helping prevent drug overdoses and reducing harm is our priority.

SAFE™ Harm Reduction DOA Test QuickProfile™ DOA Wipe Test

For the qualitative assessment of illicit drugs in environmental samples.

Effective tools for professionals involved in the harm reduction and the fight against the use of illicit drugs, including public health officials, medical professionals, harm reduction workers, addiction specialists, mental health professionals, law enforcement officials, social workers, pharmacists, and community outreach workers.

Fentanyl • Xylazine • Opiates

& other illicit drugs



Available as single or multidrug test

Simple to use **A**ffordable cost **F**ast result **E**ffective Harm Reduction



www.QuickProfileDOA.com



www.LumiQuick.com


SAFE™ Harm Reduction DOA Test

Simple to use **A**ffordable cost **F**ast result **E**ffective Harm Reduction

	QuickProfile™ Harm Reduction Tests	Other Tests in the Market
Simple to Use	<ul style="list-style-type: none"> All in self-contained device to avoid user and surface contamination. Sample collection device tube can preserve the sample for further lab testing. Test can be used for both solid and liquid samples. 	<ul style="list-style-type: none"> User needs to purchase micro sample scoop. User needs to find the container and do subjective measurement of water for the testing. Can only be used for solid powder samples.
Affordable Cost	Made in the USA, directly from manufacturer or appointed distributors to lower the product cost.	Manufactured from overseas with expensive freight and retail price.
Fast Results	Results can be obtained in 3 minutes with simple collection procedure.	Results within 5 -10 minutes plus time time-consuming sample collection procedure.
Effective Harm Reduction	High sensitivity Fentanyl: 100 ng/mL Xylazine: 500 ng/mL	Fentanyl: 200 ng/mL or not specified Xylazine: 1000 ng/mL or not specified

Simplified procedure

1. Collect the sample
2. Elute and homogenize the sample in the buffer
3. Dispense 3 drops into the well «S»
4. Wait for 3 minutes and read the results

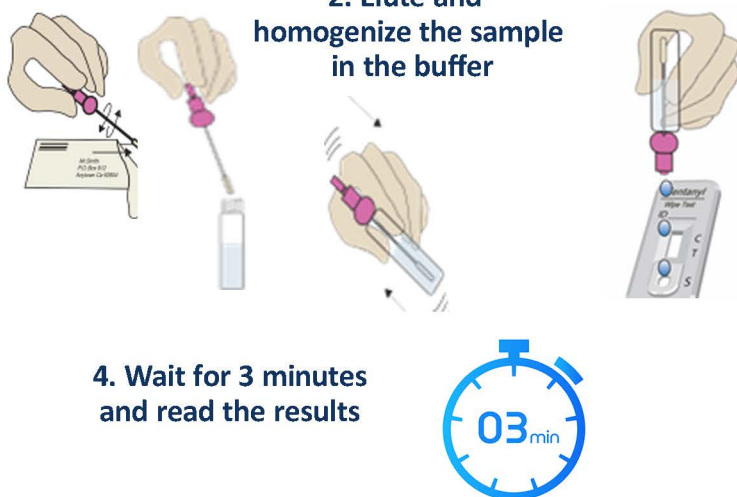


Results interpretation:


PRESUMPTIVE NEGATIVE:



PRESUMPTIVE POSITIVE:

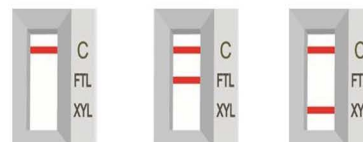

INVALID:

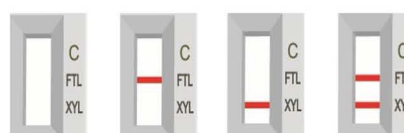

This product only provides presumptive results, does not assess the quantity, purity or safety of the substances tested

For more information on other drugs available or ordering, contact us:

Tel.: +1(408) 855-0061 - purchasing@lumiquick.com

www.QuickProfileDOA.com

www.LumiQuick.com