

GREEN DIAMOND




  
AXICON  
TECHNOLOGIES

## PRINCIPLE

Green Diamond™ utilizes 10 x 532nm true green laser beams, which are monochromatic, collimated, and coherent. These lasers create temporary pores in fat cells, allowing fatty liquids to escape, which are then naturally flushed out through the lymphatic system.

Unlike traditional methods like fat freezing that focus on eliminating fat cells, Green Diamond™ Laser trains fat cells to shrink and behave like healthy cells, promoting positive hormone responses.

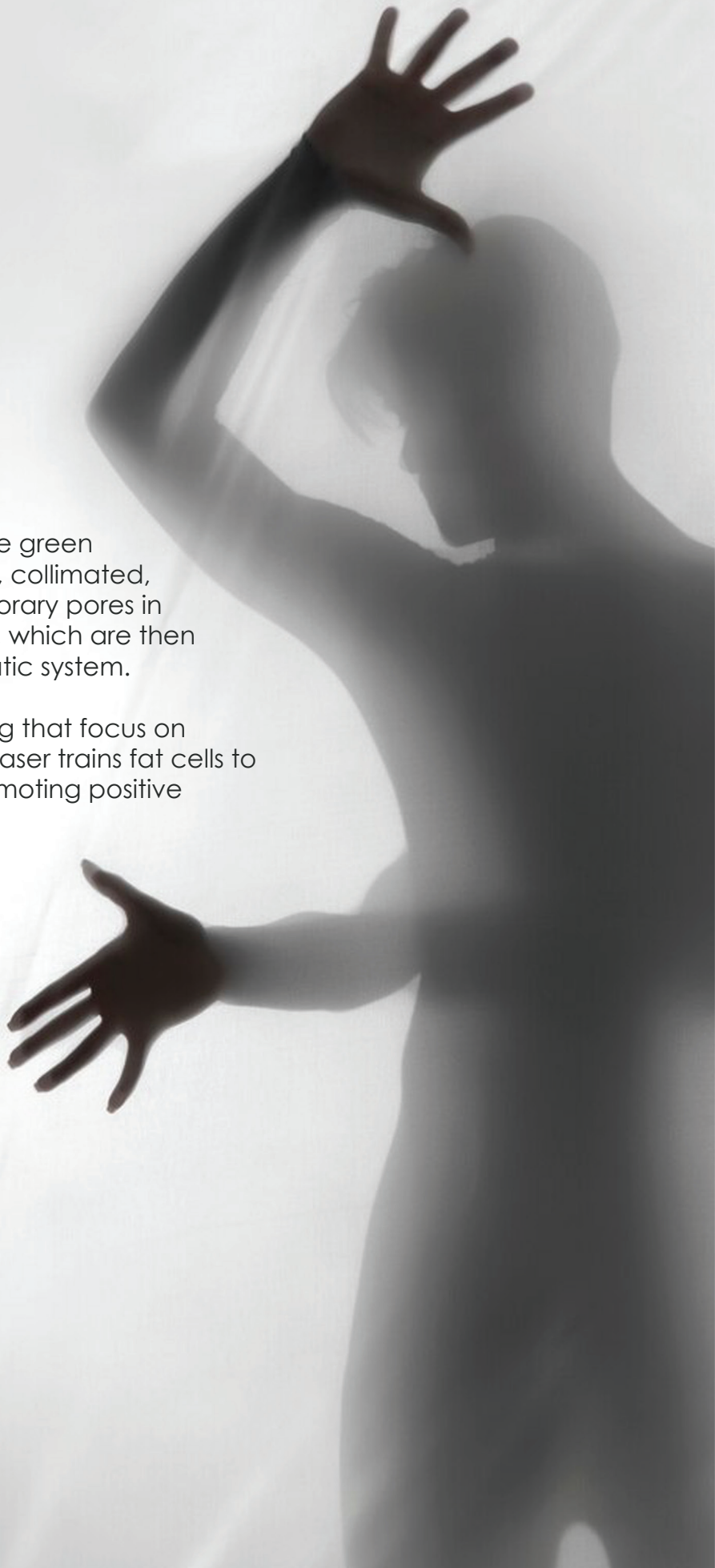



While there are no regulated contraindications, caution is advised for pregnant women, individuals with pacemakers, or those with photosensitive epilepsy due to a lack of long-term evaluations. Additionally, using the device over areas with known cancer is not recommended.

Results from **Green Diamond™** are non-temporary, provided clients maintain a healthy lifestyle. However, if clients exceed their daily energy intake, they may regain inches lost during treatment. Therefore, promoting small lifestyle changes alongside the course is recommended, with the option of maintenance treatments if necessary.



GREEN DIAMOND

# 10D ROTATING GREEN LASER LIGHTS





Each Green Diamond™ session lasts just **30 minutes** and is fully automated. Patients typically see improvements after the first session, with continued enhancement throughout subsequent treatments. The number of sessions required varies based on individual goals, lifestyle, medical history, and medication usage, with most patients undergoing 6+ sessions spread over one, two, or three treatments per week.

Green Diamond™ effectively reduces overall body circumference while providing the flexibility for clients to target specific areas of stubborn fat and cellulite. While many individuals concentrate on their midriff and thighs during treatments, the device is versatile and can target various areas of subcutaneous fat, such as the bra strap area, upper arms, jowls, knees, and more.



**Is there any restriction during treatment?**

During treatment, you can continue your normal activities. It is highly recommended to follow the treatment plan by staying hydrated, exercising, and eating a healthy low-carbohydrate diet.

**Will it rebound?**

Once the fat leaves your body, it only returns if you consume more calories than you burn. An improper diet and excess calorie intake will naturally cause fat accumulation.

**Who is suitable for the treatment?**

This treatment is suitable for most people who want to lose inches in the hips, waist, and thighs. It is also ideal for individuals looking to jump-start a new diet and fitness regimen.

**Is it safe?**


Yes, cold laser technology is FDA-approved. It can safely and effectively emulsify fat and reduce fat in the hips, waist, and thighs.

**SAFE****FAST**



SPECIFICATIONS

MAXIMUM REACH OF LASER HEAD	110cm
ANGLE ADJUSTABLE OF LASER WINGS	100 degree
WEIGHT OF LASER HEAD	12KG
SIZE OF SCREEN	12.1 inches
POWER OF DIODE	200mw
WAVELENGTH OF DIODE	532nm
VOLTAGE	90 - 240V
POWER	120W

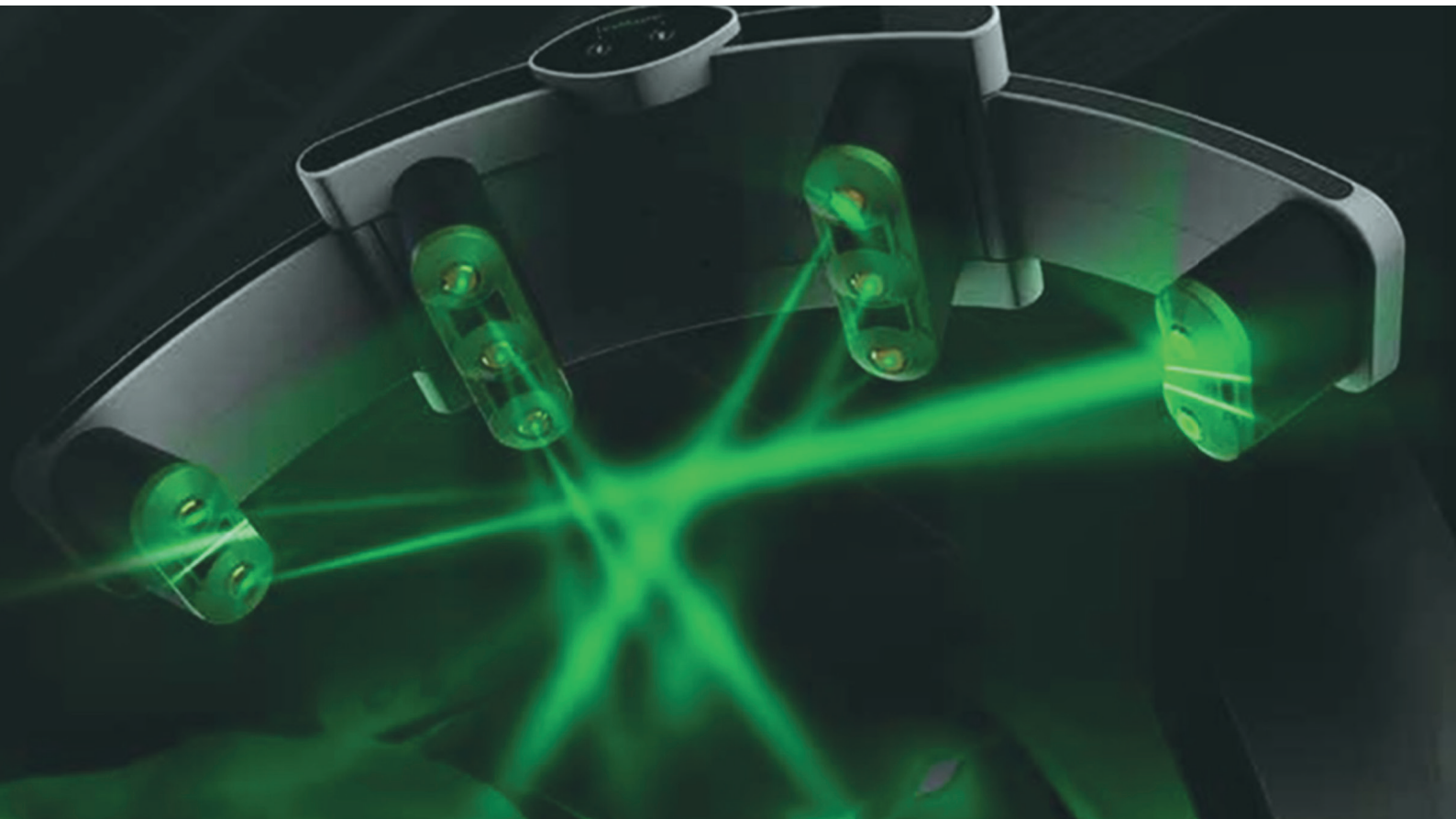



**Tomorrow's Technologies.  
Today.**

**[www.axicontechnologies.com](http://www.axicontechnologies.com)**

© 2024 Axicon Technologies LLC - USA  
All rights reserved.