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INSTRUCTIONS FOR USE GLASS FIBRE POST

EN

1. Remove the guttapercha with the usual instruments.
2. Choose the post size according to clinical evaluation and x-ray. Only for sterile posts. Do not use if the blister is damaged.
3. Determine the post length (remember the guttapercha seal in the 1/3 apex).
4. Get the root canal ready with the appropriate calibrated drill.
5. Try the post in the root canal to determine the length.
6. Cut the post at a low speed with a diamond cutting disc. Never use tweezers as they will cut the fibre in the post.
7. Clean the post surface with alcohol to remove impurities.
8. Etch the root with the etching gel.
9. Rinse the root canal well, taking care to remove all the etching (use of an endodontic irrigation needle is advised).
10. Dry the root canal with paper cones, leaving the surfaces damp. 11. Apply the adhesive in the root canal (according to manufacturer's instructions).
12. Light cure the adhesive.
13. Apply the adhesive on the post.
14. Light cure the adhesive.
15. Inject the cement into the root canal.
16. Insert the post in the root canal, pushing it gently, spread the surplus cement around the post and keep it still until completely hardened.
17. Only for glass fibre post; light cure for a minimum of 60".

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BOX: PLASTIC (C/PS81)

SEPARATE WASTE COLLECTION

Check the provisions of your Municipality

THERMOFORMED: PLASTIC (PS 6)

SEPARATE WASTE COLLECTION

Check the provisions of your Municipality

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EN

Intended use:

The endodontic bur is an instrument used to prepare the root canal to the insertion of the corresponding post.

Product handling

All the instruments produced must be handled only by medical personnel with surgical gloves and surgical equipment previously sterilized.

(if supplied in kit) The products must be stored in their special sterilizable surgical boxes.

For what concerns drills, the maximum number of working cycles in which they can be used is 30. An exceeding use can damage the patient, due to a loss of drill performance.

Instructions for Use:

- 1) Before being used, the drills must be disinfected, cleaned and sterilized
- 2) Place the instruments in the appropriate disinfection trays; use non-corrosive disinfectants
- 3) Clean the instruments in an ultrasonic bath
- 4) Dry tools
- 5) Before sterilization, place the instruments on suitable supports
- 6) Sterilize by moist heat at 134 °C for 5 minutes (2 atm) with cycles compliant with the requirements of ISO 17665-1.
- 7) After sterilization, store in a dry and clean place, using airtight containers

- 8) Use the burs on the contra-angle handpiece
- 9) Use plenty of water spray during all the phases of using, for a good cooling
- 10) Avoid levering on the cutter while using
- 11) Respect the maximum speed of use
- 12) Maximum speed of use 15,000 rpm
- 13) The tools must be kept in perfect working order; damaged or particularly worn cutters must be replaced; it is recommended not to exceed 30 reuses.
- 14) For the temporary storage of the exhausted DM, use the special containers for hospital waste; for disposal, follow the Waste Management Regulations issued by the local managing body (municipality, etc.), in relation to special hospital waste.

ENVIRONMENTAL INFORMATION PACKAGING

BOTTLE: PLASTIC (PP05)
SEPARATE WASTE COLLECTION
Check the provisions of your Municipality

CAP: PLASTIC (PE-LD04)
SEPARATE WASTE COLLECTION
Check the provisions of your Municipality

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AVENIR Srl