



MyView 24\"/>

User Manual

Index

Safety Tips.....	1
Components	2
Standard Fittings	2
Button Characteristics	3
Instructions	4
Installation	4
Basic Using and installation	4
Function	5
Power On/Off	5
Magnification Adjustment	5
Mode Toggle	6
Line Marker/Mask	6
Brightness/Contrast Adjustment.....	7
Focus lock Function	8
Find Function	8
X-Y Reading Table	9
Specifications	10
Warranty.....	11

Safety Tips






For your safety, please carefully read the notice below:

- **Please keep the device dry, away from moisture, rain, dust, and extreme temperatures.**
- **Do not place the device close radiators and heaters, or in direct sunlight for a prolonged period of time.**
- **Do not place the device beside flammable fluids.**
- **Do not attempt to repair the device yourself. Any damages caused by dismantling or maintaining the device by users will void the device warranty.**
- **Ensure that the device is placed on a stable surface to prevent it from damaging or causing injury to its user.**
- **The appropriate operational temperature of the device is 32 degrees F to 104 degrees F.**


Components

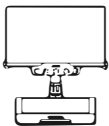Standard Fittings

Main body	1 pc
User Manual	1 pc
Internet Connectivity Wire	1 pc
MINI HDMI to HDMI Wire	1 pc
LCD Power Wire	1 pc
Power Adaptor and Power	1 pc
Wire Cleaning Cloth	1 pc

 Main Body	 User Manual	 Ethernet Cable
 MINI HDMI to HDMI Wire	 LCD Power Wire	Power Adapter and Power Cord

Button Introduction

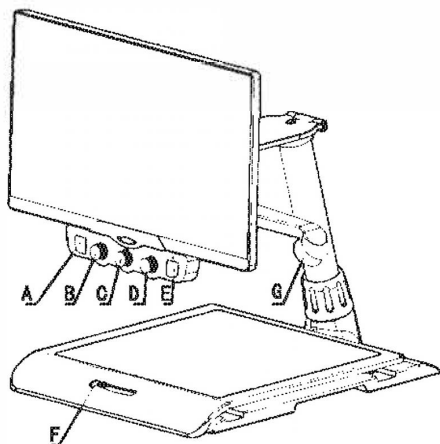

- A. Find Button**
- B. Brightness Adjustment Knob**
Focus Lock Function (Writing Mode)
- C. Mode Toggle and Magnification Adjustment Knob**
- D. Line Marker/Mask Selection and Adjustment Knob**
- E. Power On/Off Button**
- F. Focus Lock Button**
- G. LED Light**

Instructions


Installation

- Installation requirements:
 - √ AC Power Supply
 - √ Clean and Neat Table
 - √ Adequate Lighting
- Take out main body and wires and place it on a stable table surface
- Connect Wires
 - √ Connect Internet Wire to Main Body and Button Controller
 - √ Connect the device to a power supply socket
 - √ Connect the device monitor to the HDMI inlet using the MINI HDMI to HDMI Wire
 - √ Connect the DC LCD and the Monitor Power Access using the LCD Power Wire


Please remove the protective film of the camera at the first use.

Function Introduction

Power On/Off

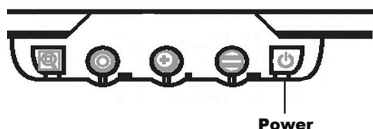

Connect the device to the power supply socket using the Power Adaptor and Power Wire. The device enters a standby mode when a Red Power Light appears. To enable the device's working status, press the Power ON/OFF button on the right-hand side of the control panel. A green LED light will appear.

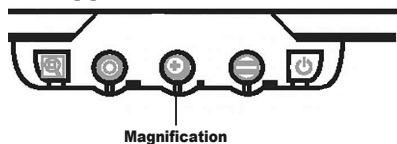


Magnification Adjustment

The Magnification Adjustment Knob is located in the center of the control panel. Rotate it clockwise to zoom in (increase) the magnification; rotate it counter-clockwise to zoom out (reduce) the magnification .

MyView 24" HD Desktop Video Magnifier

Mode Toggle Knob

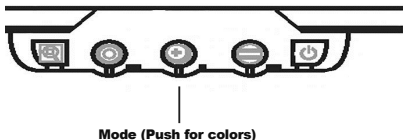


Mode (Push for colors)

The Mode Toggle Button is located in the center of the Magnification Adjustment Knob. Press it repeatedly to switch between the following eight different successive display modes:

- a) True color
- b) Grayscale
- c) Black text on White background
- d) White text on Black background
- e) White text on Blue background
- f) Yellow text on Black background
- g) Yellow text on Blue background
- h) Green text on Black background

Line Marker/Mask Feature

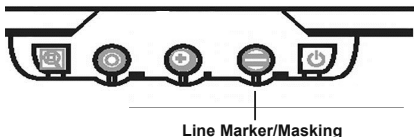

Line Marker/Masking

The Line Marker/Mask Button and Knob is located on the right of the control panel. Press the Line Marker


MyView 24" HD Desktop Video Magnifier

button to display:

- a) Horizontal line marker
- b) Vertical line marker
- c) Horizontal line mask
- d) Vertical line mask
- e) Display no line


While using the Line Marker/Mask feature, rotate the knob clockwise to widen the distance between the Line Marker/Mask, or rotate it counter-clockwise to narrow it.

Brightness/Contrast Adjustment

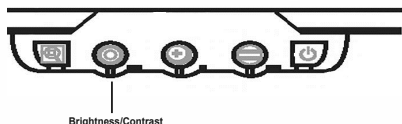


The Brightness/Contrast Adjustment Knob is located on the left side of the control panel. The brightness and contrast levels of the image can be adjusted to get clearer and higher-contrast images.

Focus Lock Function

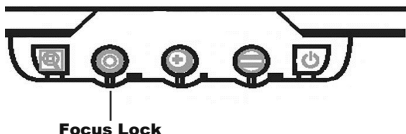

The Focus Lock button is located on the left-most side of the control panel. When functioning normally, the camera will use an automatic focus mode. As the button is pressed, a "Focus Lock" words will be displayed on the upper left corner of the screen, and the automatic focus will be disabled. Press it again to re-enable the automatic focus; the "Focus Lock" words will disappear.

Find Function

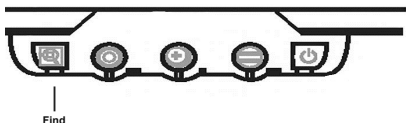


Press the Find Function Button located on the left-most side of the control panel, to reduce the magnification to its minimum and use the "+" locator that will appear on the center of the screen to reposition the reading document appropriately. When the document is centered as desired, release the Find button so that the magnification returns to its previous level.

X-Y Reading Table

Users can move the text and images horizontally or vertically by moving the X-Y reading table. When the mechanical lock is positioned to the left (see Figure A), the reading table can move freely. When the mechanical lock is positioned to the right (see Figure B), the reading table is locked into place. For your safety, keep your hands on the reading table's navigation plane as the it is repositioned as desired.

When the X-Y Reading Table is locked, do not push it hard, it will cause the X-Y Reading Table locking invalid.

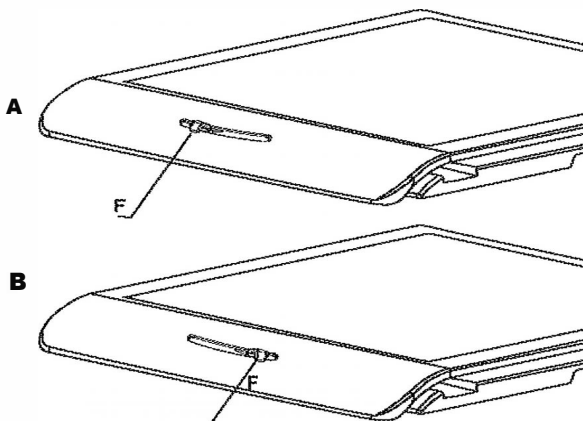

MyView 24" HD Desktop Video Magnifier

Specifications

Monitor	24" NOMINAL	LED wide screen with ultra narrow frame design and clear image quality
Button		1. POWER ON/OFF Button 2. Mode Toggle djustment Knob 3. BrightnessAdjustment Knob and FOCUS LOCK Function (Writing Mode) 4. Line Marker/Mask Selection & Adjustment 5. FIND Button
Movement		Handle cutouts on base platform for carrying
Power	Support	Input: 100-240VAC
Control Panel light	Support	LED light 2 colors Red light: Power Off; Green Light: Power On
Screen	Support	1. Monitor: 1080P 2. High Definition contrast
Magnification	Support	2.5X-70X
Multi-Mode	Support	8 modes: true color, grayscale, black text on white background, white text on black background, white text on blue background, yellow text on black background, yellow text on blue background, green text on black background
Brightness	Support	7 levels
Memory	Support	Under Normal shutdown condition, Memory Function maintains the last magnification, mode and brightness level
FOCUS LOCK	Support	Can be used for readjusting reading comfort
Line Marker/ Mask	Support	Line Marker/Mask adjustment
Foldabla design	Support	Package dimension foldable (with monitor): 650x550x345 mm NetWeight:15.8 Kg
Locator	Support	Find Button

MyView 24" HD Desktop Video Magnifier

TWO YEAR LIMITED WARRANTY

Patriot Vision Industries LLC. warrants the MyView 24" HD, effective from the date of delivery, to be free of any defects in material and workmanship for the original owner listed on the inserted "Warranty Registration Form".

The MyView 24" HD has been designed for the individual purchaser and is to be used in a stable climate controlled environment (refer to page 1 in user manual), and is non-transferable to secondary users, groups or multiple users.

WARRANTY PERIOD: For a period of two (2) years from original purchase date of the product, Patriot Vision Industries LLC. will, at its option, repair or replace with new or refurbished product or parts, any product or parts determined to be defective. This warranty does not cover service calls and/or shipping and handling fees to Patriot Vision Industries LLC. Ocala, Florida.

WARRANTY EXCEPTIONS: Patriot Vision Industries LLC. does not cover defect or damage caused by modification, alteration, repair, or service by anyone other than an authorized service center (Patriot Vision Industries LLC.). Abuse or misuse; operation contrary to the instructions provided with it; abnormal operating conditions or events including but not limited to lightning, fire, flood, earthquake, fluctuations or surges of electrical power or Acts of God, damage due to improper packaging or transit by someone other than Patriot Vision Industries LLC., or the use of any replacement parts other than those approved by Patriot Vision Industries LLC.

OBTAINING WARRANTY SERVICE: NOTIFY Patriot Vision Industries LLC. toll free at 800-364-1608. Once qualified for warranty service, a return authorization (RA#) along with shipping and/or on-site service instructions will be given. Any equipment returned and shipped to Patriot Vision Industries LLC. must be returned in the original shipping container(s). The individual user will be responsible for all costs associated with returning any product for warranty service. Patriot Vision Industries LLC. will be responsible for the cost of shipping (standard ground) the remedied product back to the individual user.

Any parts or products replaced under this limited warranty will become the property of Patriot Vision Industries LLC.. This limited warranty is only valid in the United States.



Contact your authorized Patriot® Dealer or us at:
800-364-1608 / 352-203-4164

Fax: 352-203-4617

Website: www.patriotvision.com

Email: info@patriotvision.com

Support Email: support@patriotvision.com

Patriot Vision Industries, LLC, is an affiliate of Low VisionAids, Inc. Patriot, Magnifying America, the words and their symbols/logo/mark are licensed trademarks owned by Low VisionAids, Inc. Copyright 2026. All Right Reserved. Designed & Assembled in USA. Please be advised that this user guide may contain errors, and is therefore subject to being changed or updated without prior notice.
