


The ultimate tube lube
non-drying, non irritant
and film forming

TrachyCalm

2% Aqueous solution Sodium Hyaluronate

- Aqueous endotracheal tube lubricant
- Unique biodegradable single-use applicator
- Formulated for intubation and extubation
- Well tolerated in all species

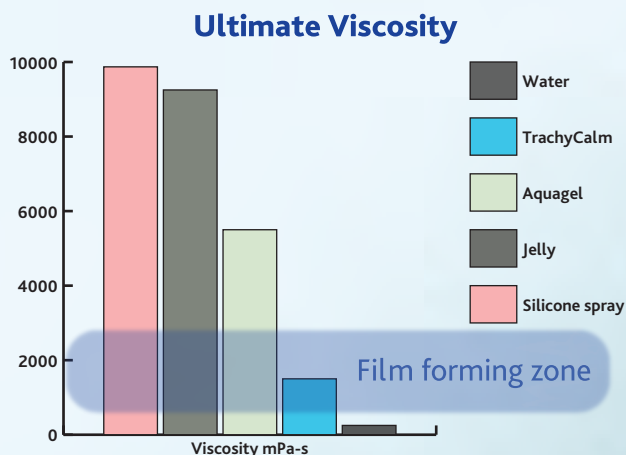



Special introductory offer:

**£14.00 per bottle
+ 40 biodegradable
applicators**

Directions for use:

Prime pump prior to first use.

Insert application straw into nozzle.

Apply several pumps to endotracheal tube and spread evenly with side of straw or gloved finger.

Post operatively insert straw into oral cavity and apply two pumps at 3 and 9 o'clock position prior to ET tube removal.

Remove straw from bottle and place in normal waste (straw is biodegradable).

Ingredients:

Aqua, Sodium Hyaluronate,
Sodium Benzoate

Quantity Required:

Name:

Practice Name:

Address:

Postcode:

Email:

Wholesaler:

phone: 01403 791313

email: mail@invictavet.com

Further information is available on request from:

Invicta, Unit 6 Shiprods Farm, Bashurst Hill, Itchingfield, Horsham, West Sussex RH13 0PD

T: 01403 791313 F: 01403 790062

E: mail@invictavet.com www.invictavet.com

INVICTA
ANIMAL HEALTH