




IHG BIOSCIENCES

Shipping Protocol- IHG Shipping Kit

On request, IHG Biosciences provides pre-addressed (postage paid) shipping kits that accommodate up to 20 glass slides and/or FFPE specimen blocks.

This kit includes (as pictured):

UPS Laboratory Pak, Two Specimen Bags, Two Styrofoam Slide Mailers, One IHG Biosciences Specimen Transport Box, and Printed Instructions.

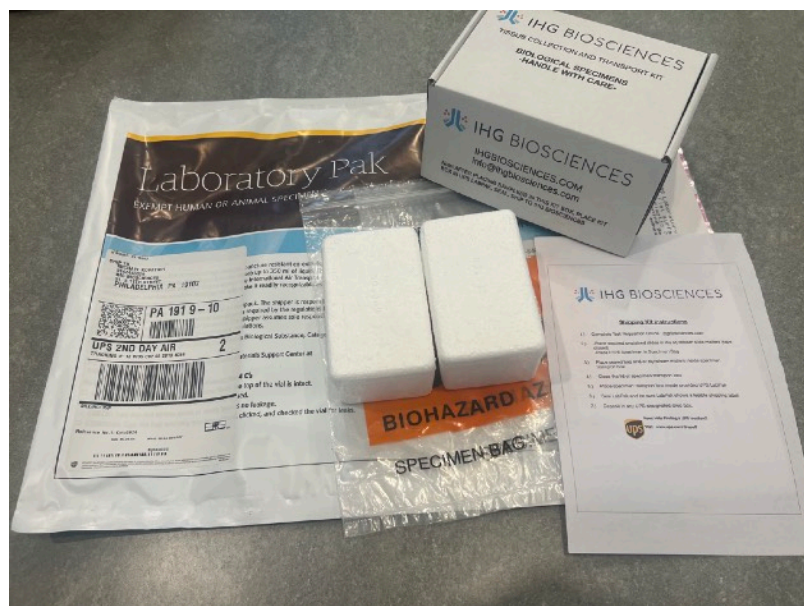

Included instructions indicate that a requesting physician or laboratory should first fill out and submit the requisition form on ihgbiosciences.com

Specimen slides and/or specimen blocks are packaged in the provided styrofoam mailers and are enclosed and sealed into provided specimen bags.



These are then packed in the crush-resistant IHG Specimen Transport Box. With tabs folded in and closed, the box does not need to be taped.



The Specimen Transport Box can then be inserted into the provided UPS LabPak. The LabPak has a self adhesive closure that will seal the specimen box inside for safe shipment.

Sealed within the LabPak, the package can be deposited to any approved UPS drop box or facility.

