Please have your Bluetooth ON before taking your test. Your device will connect AUTOMATICALLY. You DO NOT NEED TO MANUALLY PAIR your device, please avoid manually pairing your device, because if you do, your device will be unable to connect to your phone, and you will need to replace it.

Instructions to take your test:

1. Open application
2. Select region
3. Username: firstlastname (PLEASE CONFIRM WITH OFFICE)
Password: sober
* all lowercase no spaces *
4. Turn on breathalyzer (hold grey button for 3 seconds until blue light shows)
5. Tap green session on screen
6. Allow all permissions
7. Breathalyzer will connect automatically in the app, HOWEVER the first time it might take longer to connect, I would recommend to not wait more than a couple of seconds and just close your app and reopen (you will NOT need to sign in again) and press green session again. From then on, if Bluetooth is on BEFORE you press on green session, device will automatically connect in a couple of seconds.
8. Press on “Take test” at top right corner.
9. When “BLOW NOW” appears on your screen, blow into your device with a steady, firm, and consistent breath (as if trying to keep a candle flame steady with your breath) for 5 to 6 seconds until the device clicks. Please avoid blowing as hard or low as possible.
10. When you see your picture on the screen, press “Submit Results” and wait for main screen to appear again.
11. Refresh application by pressing arrows at top right corner.

Make sure the test results have been uploaded before the window closes. For example, if you are taking your test and the time is 8:00am while it is still loading the test will show as **MISSED**.

Always refresh the application after submitting the results

If the test disappears after you submit the results it PASSED.

If the test returns **RED** that means it detected alcohol so you must **RETEST** every 15 minutes until it disappears from the home screen.

Please charge the device every 3 days for about 7 hours. If you ever accidentally forget to charge the device, it will work while it is plugged in. Turn on the device, plug in the charger, restart the test page.

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