


# How to Test Step by Step with Android

## 1. Open SoberSense Application



## 2. Enter Login Credentials

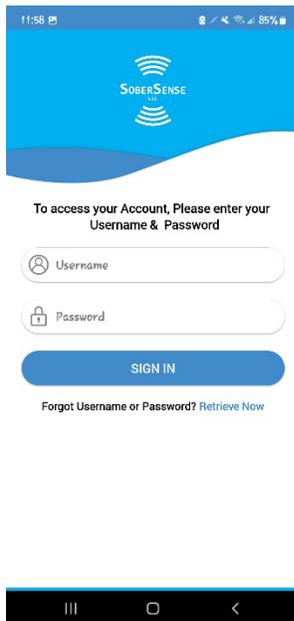

Username: firstnamelastname

Password: sober

All Lowercase with No Spaces

3. Turn on Device by holding the gray button down for about 3 seconds. You will not need to touch the button again until after your session has been submitted. Hold again for 3 seconds to turn it off.





# How to Test Step by Step with Android

4. You can now select your session. You will do this by tapping on the green session



**5. At this point the breathalyzer will start to connect automatically. You will NEVER have to put a pin to connect to the device. Please click on "TAKE TEST" at the top right of the screen to proceed.**

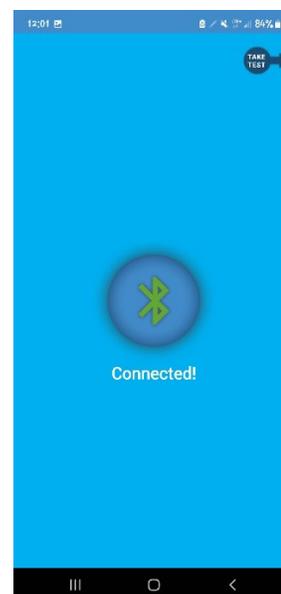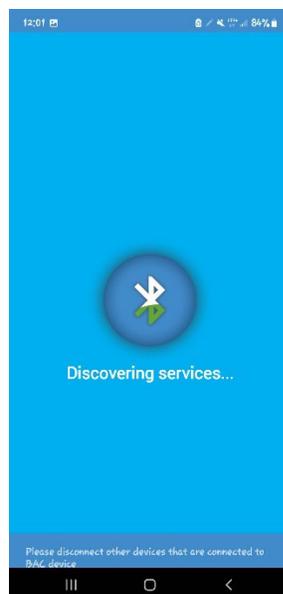

**6. "Preparing Device", "Breathe Deep", "Blow Now"**

# How to Test Step by Step with Android

7. Blow steadily for about 6 seconds until you feel a click inside the breathalyzer.

8. Once completed you will see a submit results button. Please tap on that to submit.



9. Tap on the top right refresh arrows to confirm that the session has disappeared.

