


IPHONE ERRORS

Error 1- WAITED TOO LONG TO BLOW

This will happen if you wait too long to blow after "BLOW NOW"


Esto sucederá si espera demasiado para soplar después de "SOPLAR AHORA"

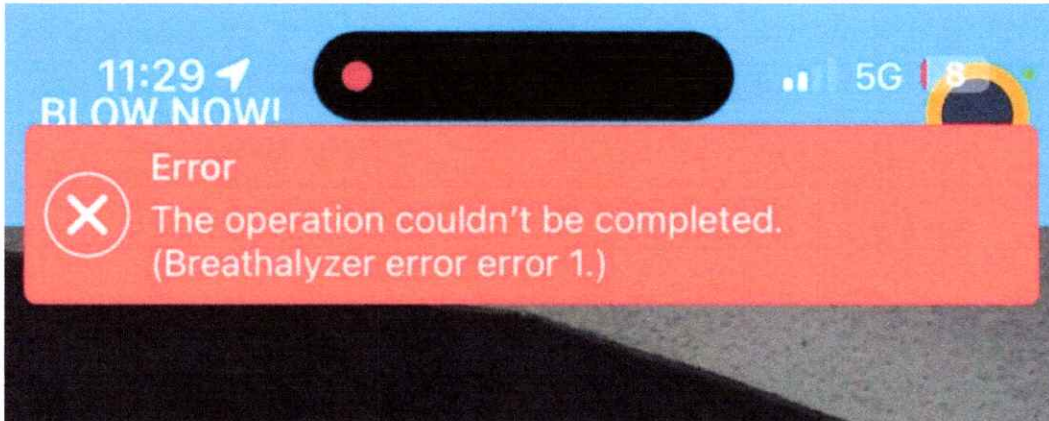

Error 2- DETECTING INHALE

This means that the device is detecting an inhale. Try again and make sure to only breathe out.


Esto significa que el dispositivo está detectando una inhalación. Inténtalo de nuevo y asegúrate de exhalar únicamente.

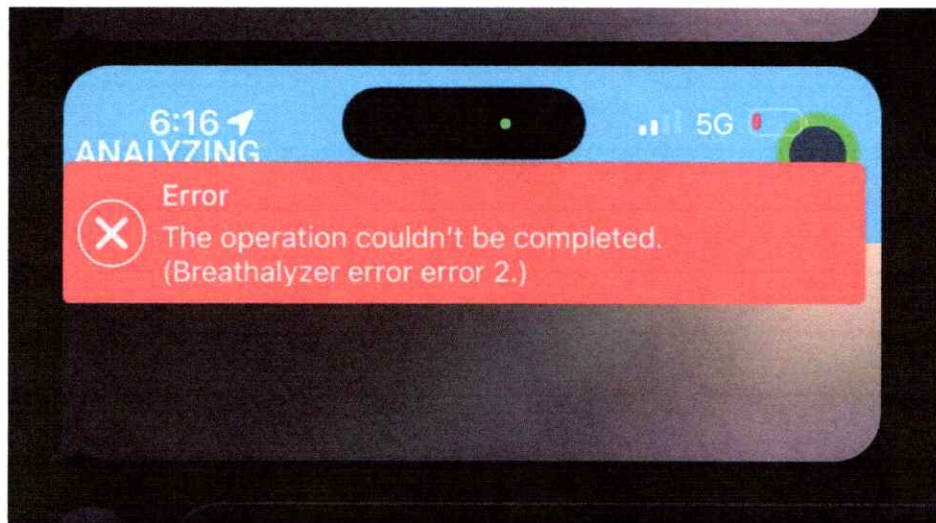

Error 3- DEVICE OVERHEATED

This means that the device has been in the heat for too long. Please put inside for 10 minutes and retry test.

Esto significa que el dispositivo ha estado expuesto al calor durante demasiado tiempo. Colóquelo dentro durante 10 minutos y vuelva a intentar la prueba.

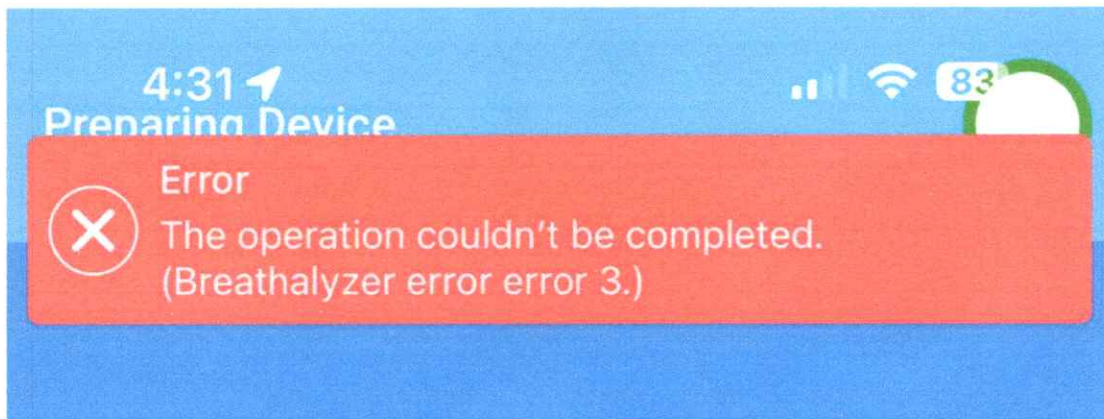

Error 4- LOW BATTERY

This means that the device has low battery. Turn on device and plug in to charger to test immediately.

Esto significa que el dispositivo tiene poca batería. Encienda el dispositivo y conéctelo al cargador para probarlo inmediatamente.

