ProneView®

Protective Helmet System

For ordering information contact Mizuho OSI
Customer Service at (800) 777-4674.

System Compatibility Guide

REF	Description	CONT	Standard Low Profile System	Adjustable Mirror System	Adjustable Helmet System	Adjustable Table System
D28501CE	Protective Helmet	1	•	•		•
D28502CE	Adjustable Protective Helmet	1			•	
D28503CE	Cushion Insert, Large	10/cs	•	•	•	•
D28505CE	Cushion Insert, Medium	10/cs	•	•	•	•
D28507CE	Cushion Insert, Small	10/cs	•	•	•	•
D28510CE	Adjustable Helmet System	1 Kit			•	
D28520CE	Standard Low Profile Mirror Platform	1	•			
D28580CE	Mirror Platform w/Post Holes	1			•	
D28590CE	Adjustable Mirror Platform	1		•		
D28700CE	Adjustable Table Platform	1				•
D28705CE	Adjustable Table Platform & Protective Helmet	1 Kit				•

Caution:

1. Do not reuse the facial cushions. The re-use of facial cushions may lead to microbial contamination and impact patient safety.
2. Any serious incident related to the use of this device shall be reported to the manufacturer and competent authority of the Member State in which the user and/or patient is established.
3. Do not reuse the Mirror or Helmet if there are visible signs of deterioration that may impact patient safety or device functionality.

Exclusive World Wide Distributor

30031 Ahern Avenue
Union City, CA 94587-1234 USA
Tel: 800-777-4674
Outside USA: +1-510-429-1500
mizuhoosi.com

4144 Avenida de la Plata
Oceanside, CA 92056 USA
Tel. # 1-(800)-546-4550

Protected by U.S. Patent Numbers: 7,426,763 B2, 8,261,385 B2.
The 3-D configuration of the cushions is covered by U.S. Trademark Registration No. 5,352,938. Patented under European Patent Number EP2346466B1.
Patented under Canada Patents 2404653, 2775072.

ProneView®

Protective Helmet System

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

CH REP

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland

ProneView®

Standard Low Profile System or Adjustable Mirror System

INSTRUCTIONS FOR USE:

Designed to support the head during prone position procedures, a system consists of Protective Helmet D28501CE, Mirror Platform (Standard Low Profile D28520CE or Adjustable D28590CE) and any ProneView® Cushion Insert. Do not use any part of this system separately. Read all instructions, cautions and warning prior to use. Fits most general surgery tables.

Rx Only

1. Open Cushion Insert package; for small patients, use Small Cushion Insert. Compressed cushions should be opened at least 60 minutes prior to use to ensure proper cushion expansion.
2. Fit Cushion Insert inside ProneView® Protective Helmet within tabs and align openings (*Fig. 1*).
3. If using the Adjustable Mirror Platform, ensure adjustable posts are set to the lowest position. The Standard Low Profile Mirror does not adjust.
4. While supine, place ProneView® Protective Helmet with Cushion Insert over patient's face, making sure the eyebrows are visible. Hold the helmet securely on the face during turn to prone position (*Fig. 2*).
5. After turning the patient prone onto the ProneView® Mirror Platform, recheck face and cushion for fit. Ensure face and eyes are within cushion opening and there is adequate clearance between nose and mirror. ET Tube or other devices should not be kinked or obstruct view of the eyes (*Fig. 3*).
6. Engage helmet legs with mirror platform posts (*Fig. 3*). Adjust as necessary to maintain a neutral neck position (*Fig. 4*).

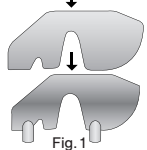
Fig.1

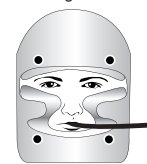
Fig.2

Fig.3

Fig.4

ProneView®

Adjustable Helmet System

INSTRUCTIONS FOR USE:

Designed to support the head during prone position procedures, a system consists of Adjustable Protective Helmet D28502CE, Mirror Platform w/Post Holes D28580CE and any ProneView® Cushion Insert. Do not use any part of this system separately. Read all instructions, cautions and warning prior to use. Fits most general surgery tables and is the recommended system for Mizuho OSI's proAXIS® Spinal Surgery Table.

Rx Only

1. Open Cushion Insert package; for patient's with small faces, use Small Cushion Insert. Compressed cushions should be opened at least 60 minutes prior to use to ensure proper cushion expansion.
2. Fit Cushion Insert inside ProneView® Adjustable Protective Helmet within tabs and align openings (*Fig. 1*).
3. Ensure adjustable helmet legs are set to the lowest position.
4. While supine, place ProneView® Adjustable Protective Helmet with Cushion Insert over the patient's face, making sure the eyebrows are visible. Hold the helmet securely on the face during turn to prone position (*Fig. 2*).
5. After turning the patient prone onto the ProneView® Mirror Platform w/Post Holes, recheck face and cushion for fit. Ensure face and eyes are within cushion opening and there is adequate clearance between nose and mirror. ET Tube or other devices should not be kinked or obstruct view of the eyes (*Fig. 3*).
6. Slide helmet legs into mirror platform post holes (*Fig. 3*). Adjust legs as necessary to maintain a neutral neck position (*Fig. 4*).

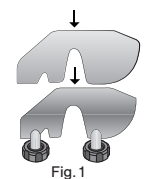
Fig.1

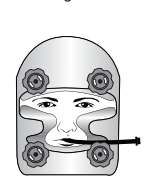
Fig.2

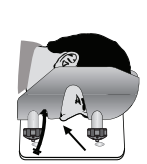
Fig.3

Fig.4

ProneView®

Standard Low Profile System with Mirror Table Platform

INSTRUCTIONS FOR USE:

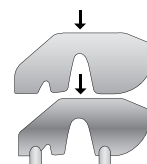
Fig. 1

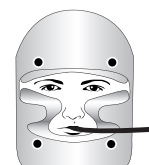
Fig. 2

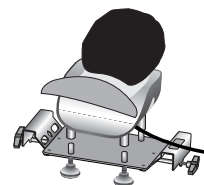
Fig. 3

Fig. 4

Designed to support the head during prone position procedures, a system consists of Protective Helmet D28501CE, Adjustable Table Platform D28700CE with Fastening Rod, and any ProneView® Cushion Insert. Do not use any part of this system separately. Read all instructions, cautions and warning prior to use. Specially designed to fit operating tables with rails such as Mizuho OSI's Spinal Surgery Top.

Rx Only

1. Open Cushion Insert package; for small patients, use Small Cushion Insert. Compressed cushions should be opened at least 60 minutes prior to use to ensure proper cushion expansion.
2. Fit Cushion Insert inside ProneView® Protective Helmet within tabs and align openings (Fig. 1).
3. Secure ProneView® Adjustable Table Platform onto rails at head of operating table. To hold Adjustable Table Platform on a table for patient rotation, use Fastening Rod.
4. Set ProneView® Adjustable Table Platform to lowest position.
5. While supine, place ProneView® Protective Helmet with Cushion Insert over patient's face, making sure the eyebrows are visible. Adjust Cephalo-caudad position on the Adjustable Table Platform so as to align helmet legs with platform posts prior to turning patient.
6. Hold the helmet securely on the face during turn to prone position, engaging helmet legs onto platform posts (Fig. 2).
7. After turning the patient prone onto the ProneView® Adjustable Table Platform, recheck face and cushion for fit. Ensure face and eyes are within cushion opening and there is adequate clearance between nose and mirror. ET Tube or other devices should not be kinked or obstruct view of the eyes (Fig. 3).
8. Adjust height of platform posts as necessary to maintain a neutral neck position (Fig. 4).

INSTRUCTIONS FOR USE**WARNINGS:**

1. Frequently monitor the patient's neck, head, eyes, nose and mouth to ensure that a safe position is maintained, and the ET tube has not been kinked or displaced. Failure to do so can lead to serious adverse consequences, including blindness, failure to ventilate, or pressure injuries to the face.
2. Failure to maintain neutral neck position can result in neck damage and/or chin and facial abrasions.
3. Some erythema may result from skin contact with the materials in this product or from compression during prolonged cases.
4. Do not use a cushion insert that has not fully expanded to the top of the Protective Helmet.
5. This product is not for continuous use of more than 8 (eight) hours.

**CAUTIONS:**

1. The ProneView® Cushion Insert is SINGLE PATIENT USE ONLY. Reuse of this pillow may result in cross-contamination. Discard after use. Covering the pillow during use does not alleviate this restriction.
2. Frequently recheck eyes, nose and neck to ensure they are in a safe position and that nothing is in contact with the eyes or eyelids.
3. The ProneView® Protective Helmet, Mirror Table Platform and Cushion Insert are to be used together and must never be used by themselves.
4. Do not use any other cushion or foam support with the ProneView® Protective Helmet.
5. Do not autoclave the ProneView® Mirror Platform or Protective Helmet. Clean per hospital protocol.
6. Replace Cushion Insert after 8 (eight) hours.

INTENDED USE:

For face and head support during medical treatment in prone position for patients under ventilatory support, including but not limited to OR and ICU. For use by medical professionals only. Not for continuous use of more than 8 (eight) hours.

INSTRUCTIONS FOR USE:

1. This item has been vacuum-packed to save shipping and storage space. After removing from package, ALLOW 60 MINUTES for the Cushion Insert to return to usable shape.
2. While supine, place Cushion Insert over patient's face, making sure the eyes are visibly centered within the pillow opening.
3. Hold the Cushion Insert securely on the patient's face while turning patient to prone position.
4. After turning the patient prone recheck face and cushion for fit. Ensure face and eyes are within the pillow opening and there is adequate clearance between nose and flat surface. ET Tube or other devices should not be kinked, nor obstruct view of the eyes.
5. Adjust patient to maintain a neutral neck position.

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