PORTUGUESE

OPTI-GARD®

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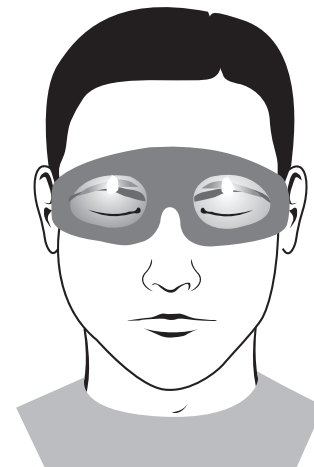
MD

CH REP

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland

Rx Only

Read complete instructions before use.

4144 Avenida de la Plata
Oceanside, CA 92056, USA
1-800-546-4550
www.dupacoinc.com

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FOR LARGE FACE USE D28300CE OPTI-GARD EYE PROTECTOR

FOR SMALL FACE USE D28200CE OPTI-GARD EYE PROTECTOR

CAUTION:

1. Do not attempt to re-use this device. The adhesive seal can become compromised after a single use and may not provide the same level of protection and functionality as the use of a new device.
2. Any serious incident that occurs related to this device should be reported to the manufacturer and Competent Authority of the Member State in which the user and/or patient is established.
3. Re-use may lead to microbial contamination and impact patient safety.

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DUPACO OPTI-GARD®

EYE PROTECTOR

Not made with natural rubber latex. – **SINGLE- USE**

Rx ONLY

Opti-Gard® is used to protect the patient's eyes from inadvertent contact during medical procedures.

DIRECTIONS

1. Remove paper backing.
2. Position over eyes ensuring that the eyes are within the openings of the lenses.
3. Press edges firmly and carefully for a tight bond.
4. Remove slowly and gently when no longer needed.

PRECAUTIONS

1. Do not use on a patient whose eyes do not fit within the openings of the lenses.
2. For patients with a small face use Small Opti-Gard® P/N D28200CE.
3. This device is not designed to bear weight. Do not place objects on the eye protector. Use a head support that prevents pressure from being put on the Opti-Gard® when a patient is in the prone position.
4. Frequently check to ensure there is no pressure on the patient's eyes during the use of this device and that the eyes remain within the openings of the lenses.

WARNINGS:

1. Compression of the eye protector on the eyes of the patient can lead to serious adverse effects, including blindness.
2. Some erythema may result from skin contact with the material used in this device, or from use during a long procedure.
3. Store product at 40° to 80° F.

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