

HAKO MED HORIZONTAL THERAPY TREATMENT PROTOCOLS

PROGRAM #1: POST TRAUMA, POST SURGICAL

Diagonal Scan 4,000-20,000 Hz 5 Minutes.....Healing and Repair
Horizontal Stimulation .02-100.0 Hz 5 MinutesStimulation
Horizontal Stimulation 100.0 Hz 5 Minutes.....Pain Reduction

PROGRAM #2: CHRONIC PAIN

Diagonal Scan 4,000-20,000 Hz 5 Minutes.....Healing and Repair
Horizontal Stimulation 100.0 Hz 5 Minutes.....Pain Reduction
Middle Frequency 4357 Hz 5 Minutes.....Healing and Repair

PROGRAM #3: CHRONIC PAIN

Diagonal Scan 4,000-20,000 Hz 5 Minutes.....Healing and Repair
Horizontal Stimulation .02-100.0 Hz 5 Minutes....Stimulation
Middle Frequency 12346 Hz 5 Minutes.....Healing and Repair

PROGRAM #4: INCREASING RANGE OF MOTION

Scan 4,000-20,000 Hz 5 Minutes.....Healing and Repair
Horizontal Stimulation 100.0 Hz 5 Minutes.....Pain Reduction
Middle Frequency 4357 Hz 5 Minutes.....Healing and Repair

PROGRAM #5: POST SURGICAL, MUSCLE SPASM

Horizontal Stimulation 100.0 Hz 10 Minutes.....Pain Reduction
Middle Frequency 12346 Hz 5 Minutes.....Healing and Repair

PROGRAM #6: POST SURGICAL, ROM, NEUROPATHY, BLOOD FLOW

Horizontal Stimulation 100.0 Hz 10 Minutes.....Pain Reduction
Middle Frequency 4357 Hz 5 Minutes.....Healing and Repair

PROGRAM #7: POST TRAUMA

Scan 4,000-20,000 Hz 5 Minutes.....Healing and Repair
Middle Frequency 4357 Hz 10 Minutes.....Healing and Repair

PROGRAM #8:

Scan 4,000-20,000 Hz 15 Minutes.....Healing and Repair

PROGRAM #9: ATROPHY

DiaScan 4,000-20,000 Hz 15 Minutes.....Healing and Repair

HAKO MED HORIZONTAL THERAPY TREATMENT PROTOCOLS

PROGRAM #10: ATROPHY

Scan 4,000-20,000 Hz 10 Minutes.....Healing and Repair
Diagonal Scan 4,000-20,000 Hz 5 Minutes.....Healing and Repair

PROGRAM #11: POST TRAUMA

Diagonal Scan 4,000-20,000 Hz 5 Minutes.....Healing and Repair
Horizontal Stimulation .02-100.0 Hz 5 Minutes.....Stimulation
Middle Frequency 4357 Hz 5 Minutes.....Healing and Repair

PROGRAM #12: BLOOD FLOW

Scan 4,000-20,000 Hz 5 Minutes.....Healing and Repair
Horizontal Stimulation .02-100.0 Hz 10 Minutes.....Stimulation

PROGRAM #13: MUSCLE RE-EDUCATION

Horizontal Stimulation .02-100.0 Hz 5 Minutes.....Stimulation
Middle Frequency 4357 Hz 5 Minutes.....Healing and Repair
Scan 4,000-20,000 Hz 5 Minutes.....Healing and Repair

PROGRAM #14:

Diagonal Scan 4,000-20,000 Hz 5 Minutes.....Healing and Repair
Horizontal Stimulation 3.0-10.0 Hz 10 Minutes.....Stimulation

PROGRAM #15: NEUROPATHY

Middle Frequency 4000 Hz 15 Minutes.....Healing and Repair

PROGRAM #16: NEUROPATHY

Middle Frequency 9009 Hz 15 Minutes.....Healing and Repair

PROGRAM #17: NEUROPATHY

Middle Frequency 12121 Hz 15 Minutes.....Healing and Repair

PROGRAM #18: NEUROPATHY

Middle Frequency 20,000 Hz 15 Minutes.....Healing and Repair

PROGRAM #19: MOTOR NERVE STIMULATION

Horizontal Stimulation .02-100.0 Hz 5 Minutes.....Stimulation
Vertical Stimulation 1-10 Hz 10 minutes.....Stimulation